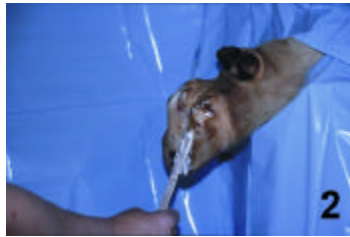
Demotec 90

Demotec[®]

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Special resins for the treatment of hooves

Demotec 90 are special resins that have been developed as expedient means for the treatment of laming cattle. Due to their outstanding quality and handy and convenient processing they are ideal problem solvers when treating diseases of the hooves. By this added height to the good hoof, the affected one is relieved of the load and put at rest. This again relieves the pain and directly reduces laming.

Preparing hoof

The good hoof will, if required, be trimmed and cleaned of dirt and loose hoof particles using hoof rasp or cutter. It will then be cleaned with spirit or alcohol, as we recommend, in order to improve bonding of the synthetic material onto the hoof. Having prepared the hoof as described above Demotec is directly applied.

This specialist resin consisting of powder and liquid is mixed at a weight-ratio of approx. 5 : 3.

Slight variations of this ratio do not markedly affect processing. To facilitate proper batching we have supplied a sufficient number of mixing beakers with relevant markings. In order to get proper batching we recommend the following procedure:

- Pour 50 ml of the liquid into one beaker (marking 1) and
- 80 g of the powder into another beaker (marking 2).

Then you pour the powder batch into the beaker containing the liquid.

Afterwards you mix the two components stirring gently, using the enclosed wooden spatula. Already after a few seconds you will get a homogeneous pulpy mass.

Processing

Directly after mixing Demotec 90 is still liquid. Approx. two minutes later it becomes pasty and can now be processed easily. Hoof sole and wooden block are both now coated with a thick layer of Demotec 90. Having pressed the wooden block hard onto the hoof sole the surplus resin will be modelled over the sides of the hoof and the wood bloc. The circumferential groove of the wood block has to be filled properly with the material to provide good stability. After about 4 – 6 minutes Demotec 90 will have set completely and when cold will withstand high loads. Due to a special combination of the ingredients Demotec 90 in pasty aggregate can be modelled into any form required, almost without being sticky. It has been established that working with soapy hands, or hands coated with protective cream, prevents the material from sticking onto the skin.

Removing the wood block

When the infected hoof has been cured completely the wood block can be removed using a chisel or another cutting tool.

Setting affected by temperature

Often animals will have to be treated outdoors (e.g. in the field). Depending on the season there will be great variances as to the ambient temperatures, which affect processability and setting markedly. Times given for processability and setting in this information refer to an ambient temperature of 21 °C. Temperatures lower than the set one will result in processability and setting being longer. Temperatures higher than the set one will result in processability and setting being shorter. As a rule of thumb we can say that a temperature of e.g. 30 °C processability and setting will be halved and at a temperature of only 10 °C they will be doubled. Because of this it should be seen to it that the material is heated in winter and cooled in summer.

General remarks

Demotec should be stored in a cool and light shielded place. Bottles and containers should be closed well after use. Contents should be protected from impurities. Store in a place well protected from children. Observe warnings on the bottles properly. Request DIN safety specifications and further details.

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