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DL Soft The new hoof trimming disc **Instructions of use**

After many years of experiences with the approved DL-Disc for professional claw treatment, today we can present a version for less trained user. The DL Soft removes clearly smaller horn shavings off the claw and so combines a good cutting capacity with the advantages of a smoothly running trimming disc with blades. That way it becomes the ideal trimming disc for all users, who do not trim claws every day by themselves.

Like the proven DL disc, the DL Soft is made of Aluminium. This together with the placing of the blades and the small distances between the blades ensures that you can work with extraordinary low vibrations and makes the handling very easy.

The 7 blades are made of hard metal (tungsten-carbide) which are very wear resistant. This ensures a good cutting capacity and a long working time. Also the titan coated longlife blades are available as spare blades.

Soft cutting combined with the advantages of a trimming disc with blades:

- Soft cutting due to the new adjustment of the blades.
- Smooth running due to 7 blades being placed in quick succession.
- Clean cut surface due to the gentle abrasion of the horn shavings.
- No dust – convenient for the user
- No heat – preserves the animal
- Blades can be used double-sided

Preconditions for the optimal use

- A good anatomical knowledge of the hoof enables the hoof trimming disc to be applied the best possible advantage. Only used properly the optimal use is possible. To learn the important things, a hoof trimming course is recommended.
- For the use of the disc a one-handed angle grinder with 750 watts is sufficient.
- To achieve the best result a speed of about 10.000 rpm is recommended.

Instructions of assembly to replace the blades

The disc will be delivered with the blades already assembled. So you directly can start with the disc. The blades only have to be replaced if there are damaged or blunt.

Each blade is fixed with a screw set. This set contains of a screw, a nut and a washer.

1. To fix the blade, please place the nut from above into the disc.
2. Then place the blade with the washer into the disc that way that the washer will be between blade and nut at the end.
3. Then place the screw into the side hole of the disc. Then the screw will be turned through the blade into the nut. Please fix it with 20 N/m.

Caution!

- Please observe the safety precautions of the grinder!
- Wear safety gloves!
- Fix the claw during application. For that, please use a proper crush.
- After some working time, tighten the screws again!
- Please clean the disc regularly!

The Demotec program for the optimal hoofcare

Demotec DL disc	10DL61
Demotec DL Soft	10DL63
1 piece spare blade	10DL64
1 complete set (7 blades)	10DL65
1 piece spare blade, titan	10DL70
1 complete set (7 blades), titan	10DL71
1 set screws	10DL67
(complete to fix 1 spare blade)	