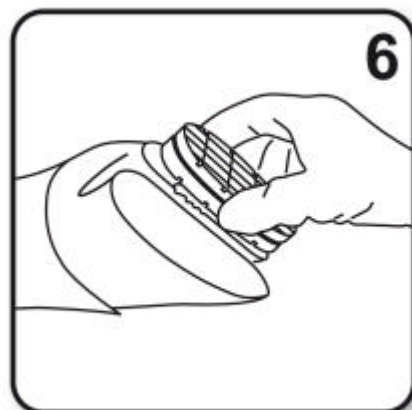
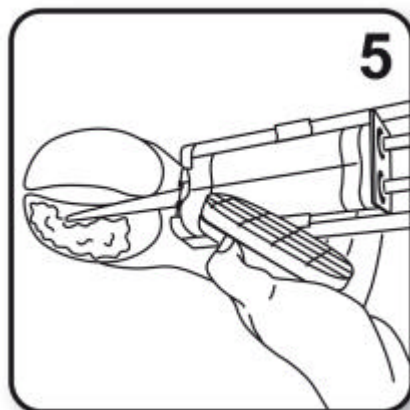
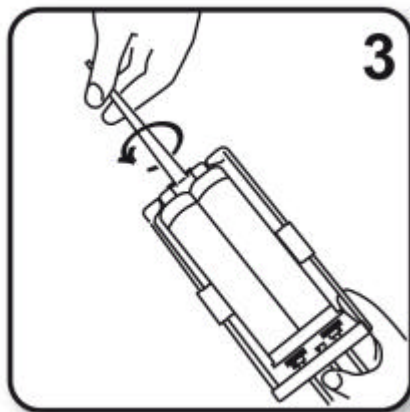
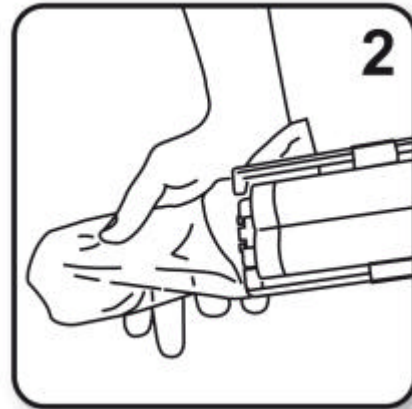
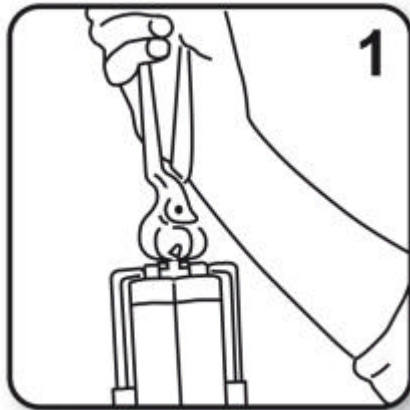
Demotec EASY BOND II

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Special resin for the hoof trimming

DEMOTEC EASY BOND II is a special resin that has been developed as expedient means for the treatment of laming cattle. Due to its convenient processing it is an ideal problem solver when treating diseases of the claws. A wooden bloc is bonded onto the good claw with the glue. By this added height of the good claw, the affected one is relieved of the load and put at rest. This relief of pressure eases the pain and directly reduces laming.

Preparation of the claw

1. Trimming of the claw
The good claw should be trimmed rather flatly and be cleaned of dirt and loose horn.
2. Degreasing of the claw
Cleaning the claw with alcohol degreases it. So the bonding of the resin is improved.
3. Drying of the claw
Moistness worsens the bonding. For drying a heating gun is recommended.

Preparation of the system

The polyurethane resin consisting of 2 components is pressed through a mixing tip and is mixed automatically.

4. Inserting and opening of the cartridge
Please insert the cartridge into the gun. After that open the cartridge with pliers. For that put the pliers on the shoulders of the cartridge and remove the stopper. Take care that you cut the stopper at the correct height. For that a stepped line is imprinted on the stopper. Then check if the opened cartridge exit is free of polymerised resin.
5. Spread out the material
Spread out some of the material. So, possible different filling levels can be equalized so that the resin will be mixed equally then.
6. Place a mixing tip
Before you start with a new treatment, always a new mixing tip has to be used.

Processing

7. Placing the resin
Put enough resin onto the claw and the wooden bloc. Take care that the last 2 cm of the bloc remain free from resin. So the pressure-sensitive back part of the claw is not bonded. Unbend now the gun to stop coming out of the material off the cartridge.
8. Place the bloc
Please press the bloc immediately onto the claw. Take care that the layer of resin between claw and bloc does not become thinner than 3 mm. Then the resin will polymerise properly.

9. After 2-3 minutes (at 20°C) the resin is polymerised and cooled down again. Now weight can be put on the bloc.

General remarks

- Temperature
The optimal temperature for processing this system is between 18°C and 25°C. If it is colder, the resin, the mixing tip and the bloc have to be warmed up. In summer maybe the components have to be cooled down.
- Preparation of the claw
The better the claw is prepared, the longer the placed bloc will adhere. Therefore the claw should be cleaned, degreased and dried carefully and should be trimmed flatly. Roughening the claw also improves the bonding.
- Tip closes the cartridge
After using, the old mixing tip can remain on the cartridge. The polymerised resin inside the tip closes the cartridge. To keep away dirt from the resin, the mixing tip should be changed only just before being used again.
- Do not touch the resin!
To avoid contact with the resin, we recommend you to wear appropriate gloves.
- If you start pressing with the gun and only one of the components or none comes out of the mixing tip, please stop pressing. Otherwise the cartridge may burst. Please unbend the gun and check the following:
 - is the cartridge opened properly?
 - are the exits of the cartridge free from polymerised resin?
 - is the mixing tip clean and free?

Removing the wooden bloc

When the affected claw is healed completely, the wooden bloc can be removed with a suitable tool like a chisel or another cutting tool.

General remarks

DEMOTEC EASY BOND II should be stored in a cool and light shielded place. Contents should be protected from impurities. Keep out of the reach of children. The warnings on the cartridge should be strictly observed. DIN safety data sheets with further detailed information are available on request.

Do not mix DEMOTEC EASY BOND II or its components with other products of the Demotec range!

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